


T₂ESC DEVELOPER SYSTEM CLEANER

A TWO-PART CONCENTRATE
USE FULL STRENGTH OR DILUTED 1:1 WITH WATER.


To Mix Cleaner:

Add 16 oz. (1/2 Bottle) Part 1 to 16 oz. (1/2 Bottle) Part 2. Add up to 32 oz. water.

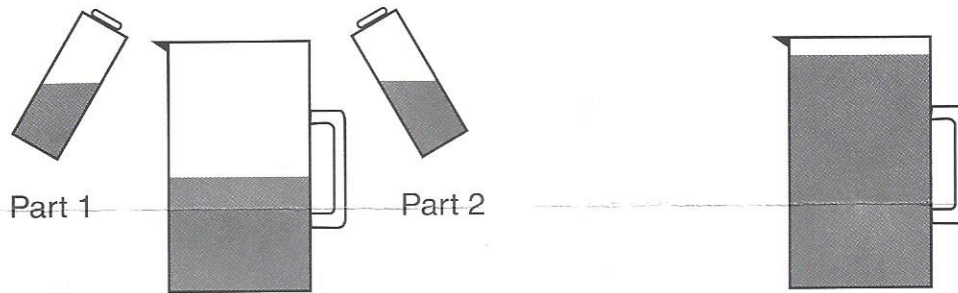

TO CLEAN DEVELOPER RACK ONLY

1. Rinse rack with water to remove excess developer.
2. While turning rollers, slowly pour mixed Cleaner over rack.
3. Dilute 16 oz. (1/2 Bottle) of Neutralizer Rinse with 48 oz. warm water.

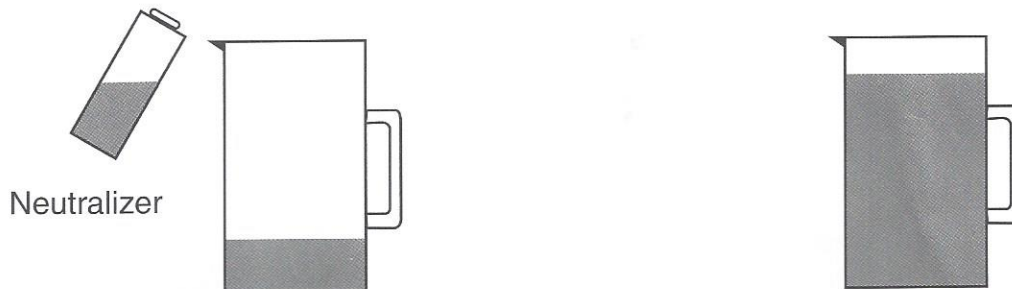

4. Pour diluted Neutralizer over the rack to complete cleaning process.

TO CLEAN THE ENTIRE DEVELOPER RECIRCULATION AND PROCESSING SYSTEM

1. Drain developer from processor and remove rack assembly.
2. Remove developer filter.
3. Rinse tank and rack with water to remove excess developer.
4. Fill processor tank with water to fill line and recirculate for 5 minutes, then drain.
5. Add 32 oz. each of Cleaner Parts 1 & 2 (1 Bottle each), to processor tank. Fill tank to operating level with warm water.
6. Replace developer rack. Turn on processor drive and recirculation. Run for 10 minutes.
7. Turn off processor and drain. Refill with warm water. Add 32 oz. (1 Bottle) of Neutralizer Rinse. Run for 5 minutes.
8. Drain processor and flush again with water.
9. Refill processor with fresh developer and appropriate starter.



WHITE MOUNTAIN IMAGING

PO Box 216 • Salisbury, NH 03268 • 603-648-2124 • FAX 603-648-2197